


Bed Bracket Assembly

HTPB


Follow the instructions below to mount the HTPBB Bed Bracket to a HTP-1500 pump. 4 screws with washers are included with the bed bracket. A "Philips" cross-head screwdriver is needed.



HTP-1500 with Bed Bracket

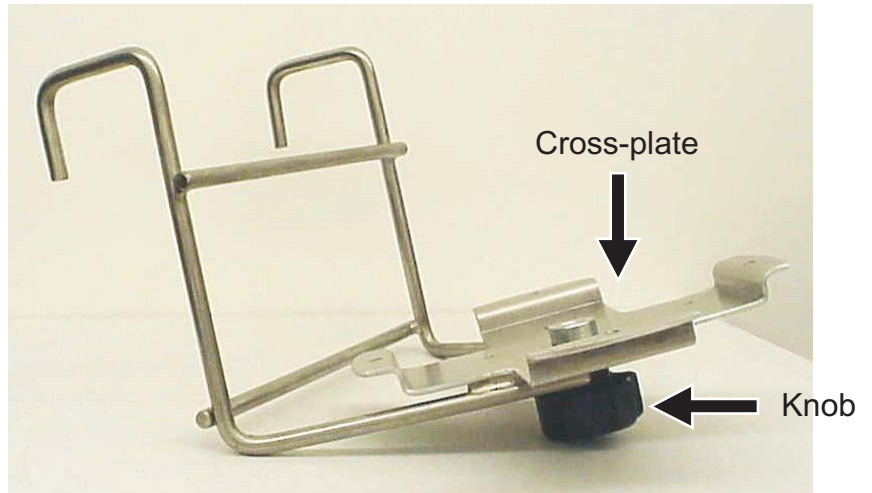

Bed Bracket

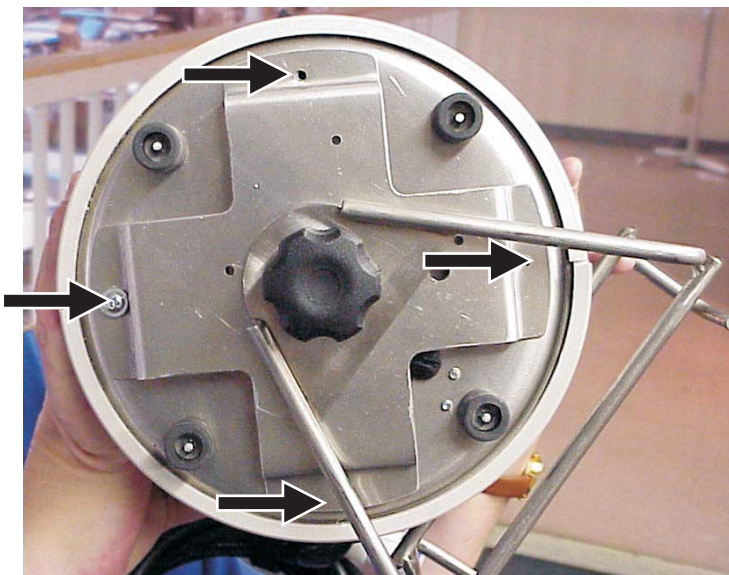

Align screw holes



Position bed hooks at backside of pump

1) Align the 4 holes in the bed bracket's cross-plate with the 4 holes in the pump's base. The bed-hooks should be at the backside of the pump. Use the 4 screws with washers to fasten the bed bracket to the pump's base. 2) Use the black knob to secure or remove the pump from the bed bracket.