

VetPad

The HTP-1500 Heat Pump works best with Adroit's "V" pads. If you don't already have Adroit pads, you can convert other pads for use with the HTP-1500. Simply use the adapters "ST-Plug" (P/N 6305-0001) included with your HTP-1500. Cut the ends of the tubing off of your existing pad and insert the ST-Plugs. See examples below.

CSZ PADS: Make a straight, perpendicular, cut in tubing **STRYKER PADS:** Angle cut tubing like below



Cincinnati Sub-Zero MaxiTherm # 273, # 274 or # 276

Blue Polymer Both Sides

(Note: "Dynax" & "Engler Engineering" pads are CSZ pads, just sold by Dynax & Engler)



Stryker Heating Pads (Formerly Gaymar)


TP-22G
Green Polymer
Material Both
Sides



TP-22B
Clear Polymer



TP-26E
Non-Woven Fabric
Material and Plastic

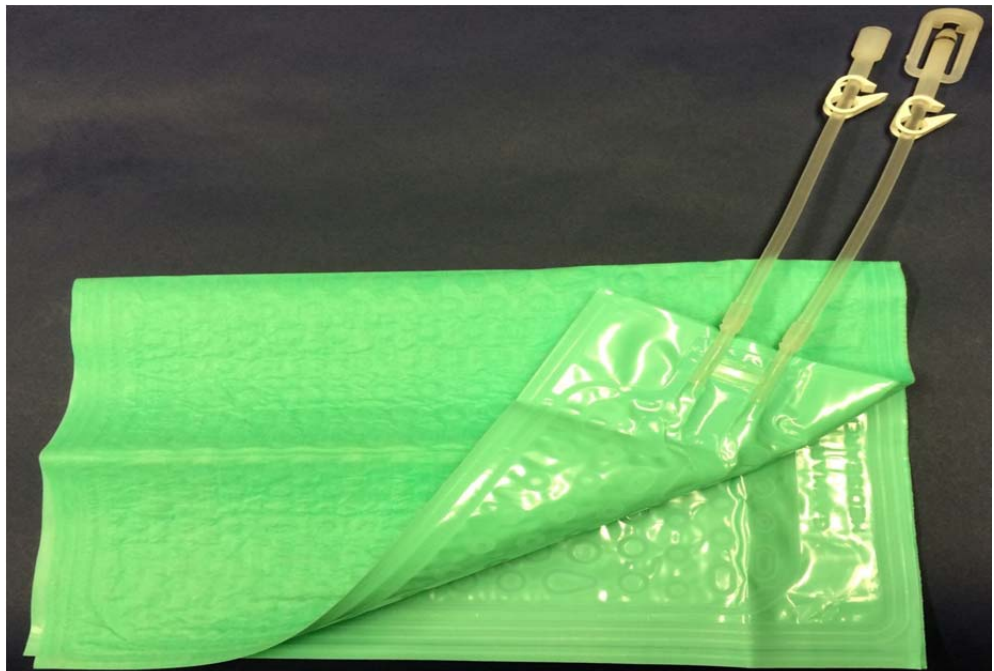

Questions?
Contact Laing Heidt
Adroit Medical Systems
Cell: 678-761-4180
laing@adroitmedical.com